

Precautions: Keep out of reach of children • Do not apply directly to the eye • Do not use if you have known allergies to any of the ingredients • For external use only • Discontinue use if you experience any redness, itching or irritation and consult your Eye Care Specialist • Single use only • For ages 3 years +




The OPTASE® family of products are designed to deliver comfort and soothing relief.




Scope Health Inc
79 Madison Av, 27 E 28th St.
New York, NY 10016, USA

Questions: +1 917 7954227
www.optase.com
hello@optase.com

OPTASE® and  are trademarks of Scope Ophthalmics.

OPTASE®
 **EYELID CLEANSING WIPES**

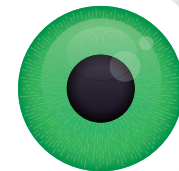

PRESERVATIVE FREE

NOURISH • HYDRATE • RESTORE

Suitable for sensitive eyes & eyelids



What is in OPTASE® TTO EYELID CLEANSING WIPES?

OPTASE® TTO Eyelid Cleansing Wipes contain Tea Tree Oil (TTO) & Hyaluronic Acid as well as other ingredients to create a unique and gentle preservative free formulation.

What are OPTASE® TTO EYELID CLEANSING WIPES used for?




OPTASE® TTO Eyelid Cleansing Wipes are used as part of a daily eyelid regimen.

Hyaluronic Acid is a natural substance found in many organs of the human body including the eye. Its special feature is that it moisturises and soothes the eyelids and can help improve eyelid elasticity.

Tea Tree Oil (TTO) is a natural essential oil from the Australian native plant *Melaleuca Alternifolia*. Our concentration of TTO was chosen to provide the optimum level of comfort.

OPTASE® TTO Eyelid Cleansing Wipes are preservative free making them ideal for daily eye hygiene.

Directions for use

		
1. Wash hands before use	2. Remove wipe from sachet and wrap over finger	3. Close eye and wipe gently and thoroughly across the eyelid and lashes several times in a side to side motion

		
4. Using a fresh wipe, repeat the process for the other eye	5. Discard the wipe after each use	6. No need to rinse after use